IMPORTANT
INFORMATION

ENAMEL plus H*Ri*

(I) ITALIANO	pag. 2
(GB) ENGLISH	pag. 3
(F) FRANÇAIS	pag. 4
(D) DEUTSCH	pag. 5
(E) ESPAÑOL	pag. 7
(P) PORTUGUÊS	pag. 8
(CZ) ČESKY	pag. 9
(LV) LATVIAN	pag. 10
(DK) DANSK	pag. 11
(H) MAGYAR	pag. 12
(GR) ΕΛΛΗΝΙΚΑ	pag. 13
(FI) SUOMI	pag. 14
(N) NORSK	pag. 15
(BG) БЪЛГАРСКИ	pag. 16
(LT) LIETUVIŲ KALBA	pag. 17
(NL) NEDERLANDS	pag. 18
(PL) POLSKI	pag. 19
(SV) SVENSKA	pag. 20

ENAMEL plus H*Ri*

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(GB) Important Information for Enamel plus HRi Universal Enamel

The new Universal Enamels (UE) have the same refractive index as the natural tooth and the same high luminosity as natural enamel. These two unique properties require the material to be applied with a technique different from any other enamel shade of composite you may be familiar with.

UE shades should be applied at the same thickness as the enamel on the tooth (or slightly thinner) that is being restored.

The more thick the layer of UE applied, the higher the value of the material (more white). A thin layer of UE will be more transparent and will enhance the blue-amber opalescent effect of the material.

If you want to further enhance the opalescence effect in the incisal area, you may use the Opalescent Blue Natural (OBN) or Opalescent Amber OA shades, for intensive white IM, IWS or IW covering these bodies with a 0,3-0,5 mm layer of Universal Enamel UE (even thinner for enhancing the intensive) as thicker layers can cover these bodies.

The New Universal Enamels and how they differ from each other

No other composite surpasses the HRi UE shades' ability to replicate natural enamel. While all of the UE shades possess the same refractive index as natural enamel, the same fluorescence and opalescent properties of natural tooth, and like natural enamel increase in value with increased thickness, there are differences between the three UE shades.

UE1 Amber hued enamel which may be used in thin layers for lower value restorations or in a thick layer for higher value restorations with amber effects.

UE2 Medium value enamel, most closely matched to natural enamel which may be used in thicker layers to obtain a high value enamel.

UE3 Very high value enamel ideal for restoring very white teeth, typically found in children or patients who have bleached teeth.

Enamel plus HRi UE shades have the same viscosity of a micro-hybrid composite, but with sculptability similar to nano-filled composite, thus making them ideal for posterior as well as anterior restorations.

NOTE: do not wet Universal Enamel with any resin or bonding because it will change the refraction index and cause the composite to become too opaque.

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