# ENA Soft

(EN) ENGLISH ENA Soft instructions	<b>Page 2</b>
(IT) ITALIANO ENA Soft istruzioni	<b>Pag. 3</b>
(DE) DEUTSCH ENA Soft Gebrauchsanweisung	<b>Seite 5</b>
(FR) FRANÇAIS ENA Soft instructions	<b>Page 7</b>
(ES) ESPAÑOL ENA Soft instrucciones	<b>Pág. 9</b>
(PT) PORTUGUÊS ENA Soft instruções	<b>Pág. 11</b>
(EL) ΕΛΛΗΝΙΚΗ ENA Soft οδηγίες	<b>σελίδα 12</b>
(CS) Český ENA Soft Návod na použití	<b>Str. 14</b>
(PL) Polski ENA Soft Instrukcje	<b>Str. 16</b>
(RUS) РУССКИЙ ENA Soft инструкция по применению	<b>Стр 17</b>
(HU) Magyar ENA Soft Használati utasítás	<b>Oldal 19</b>

ENA Soft

### **Intended purpose**

ENA Soft is a light-curing temporary filling material. ENA Soft is an elastic material for temporary restorations. It has advantages of easy removal in one piece without damage to preparation. ENA Soft is suitable also for long term temporaries because it perfectly maintains the shape of tooth. It can be used to protect implant screws.

### **BENEFITS**

- simple to apply: not sticky;
- easy to remove in one piece;
- suitable for inlay & onlay and filling on implants screws;
- strong and long lasting fillings.

### **Composition**

- Organic matrix: UDMA, HEMA
- Inorganic fillers: Nano silica
- Additives: Initiator, Catalyst, Stabilizer, Others

### **Shade**

Dentine colour (UD2)

### **Target group**

Patients of any sex (also pregnant)

Age: Children 6 - 18 years, Adults 19 - 64 years, Elderly 65 – above

### **Indications**

ENA Soft is used for:

- temporary fillings, especially on inlay & onlay
- fillings on implant screws

### **Contra-indications**

If a dry working field cannot be established or the stipulated application technique cannot be applied. Uncured resin could cause skin allergy: user should use gloves. In case of known allergy to some of the components do not use it.

### **Precautionary Information**

The product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of all allergic response, minimize exposure to uncured materials. If allergic reaction occurs, seek medical attention as needed.

### **Side effects**

Inhalation: In case of inhalation, immediately consult with a medical specialist.

Skin contact: Wash off immediately with soap and enough water, and then consult with a dermatologist.

Eye contact: Rinse the eyes immediately with enough water, and then pour clean water into the eyelids more than 15 minutes, and then consult with an ophthalmologist.

### **Materials to be avoided**

Materials containing phenolics (like eugenol) could inhibit composite curing. Avoid the use of these materials as liners.

**Users:** dentists only.

## Instructions for use

- 1) Prepare the cavity;
- 2) Application of ENA Soft;
  - a. Carefully clean and dry the cavity to be restored before applying Ena Soft. In case a cavity is without external support, we suggest to use a matrix.
  - b. Remove the cap from the syringe. Turn the extruder clockwise to dispense a sufficient quantity of material.
  - c. Apply Ena Soft in the cavity using a composite instrument.
  - d. Check occlusion and then remove excess material.
- 3) Light-cure for 10-20 seconds, increments of 4 mm thickness, using a led or halogen light-curing unit (20 sec. minimum 650 mW/cm<sup>2</sup>, 10 sec. minimum 1000 mW/cm<sup>2</sup>).
- 4) Remove the matrix if used.
- 5) The restoration can be removed with an instrument: it comes out in one piece.

## Use and storage

Do not store below 3°C/38°F and above 25°C/77°F. Shelf life at room temperature is 36 months. Ambient temperatures routinely higher than 25°C/77°F may reduce shelf life. Do not use the product after the expiration date (see label on syringe). Use the material at room temperature. Medical device, for dental use only: keep away from children. Turn back the spindle after taking out the material, to avoid sticking of the material, and close the container. After use, close container with cap and keep it closed. Avoid direct exposure to sunlight. Do not store materials in proximity to products containing Eugenol.

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